

# Saxon



**Stannah is a family-run British company which has been making pioneering products since 1860. We are the world's leading stairlift manufacturer with an unrivalled reputation for quality, integrity and service. We uphold this reputation by applying recognised standards to every product we make.**

## **Saxon**

**The Saxon is a classic yet stylish Stannah model built to ensure your safety, convenience and comfort.**

**Its sleek design and range of practical features have made it a consistently popular choice for straight stairs, and in particular for narrow staircases that other stairlifts may have difficulty accommodating.**

### **The chair**

The Saxon's compact carriage and rail, and its folding armrests, footrest and seat make it ideal for narrow staircases. When not in use, it can be folded away easily, so it takes up less space in your home and leaves the stairs clear for other users. The chair is upholstered with soft modern fabrics, which come in a range of colours.

You can even choose from several different features to tailor-make the chair to your needs. To operate the stairlift, you can either have light-touch directional buttons or a small joystick, both of which can be positioned on either arm. Additionally, whilst every seat can be manually swivelled to face the landing when you dismount, you can also opt for a powered swivel seat that will automatically turn at the top of the stairs.


Safety is always paramount, so the Saxon has a retractable seatbelt to secure you in the chair, and safety edges to stop the stairlift automatically if it meets an obstruction on the stairs. The self-charging battery means the stairlift is always ready for use, and will always work even in the event of a power cut.



Straight rail

### **The rail**

The Saxon runs along a strong rail with a slimline design and is fitted to the stairs, not to the wall, so installation is quick and clean. Straight rails are available in either silver or bronze.

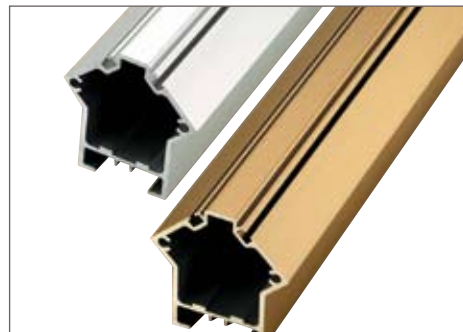

### **The service**

We pride ourselves on the reliability of our stairlifts. To give you absolute peace of mind, all new products come with a 2 year warranty and 2 free services. Reconditioned products come with a full year's warranty and first free service. We provide you with support from our nationwide teams of locally-based engineers, who are on standby 24-hours a day, 365 days a year to respond quickly in the unlikely event that you need assistance.



### Remote control

allows you to call the stairlift to you or send it to someone at the top or bottom of the staircase.



### On/Off switch

### Keyswitch

can be used to completely immobilise the stairlift, ideal if children are around



### Retractable seatbelt

will fully secure you in the chair.



### Safety edges

on the chair mean it stops automatically if it meets an obstruction on the stairs.

### Adjustable seat height

for personal comfort and ease getting off the seat.




### Upholstery material

available in modern textiles from a selection of attractive colours, so your stairlift blends in with your home.



### Arm-to-footrest link

means that when you fold the arm away the footrest folds up with it, saving you any awkward bending down, particularly important at the top of the stairs. With a fold-away seat too, the whole chair can be stored simply away against the wall.

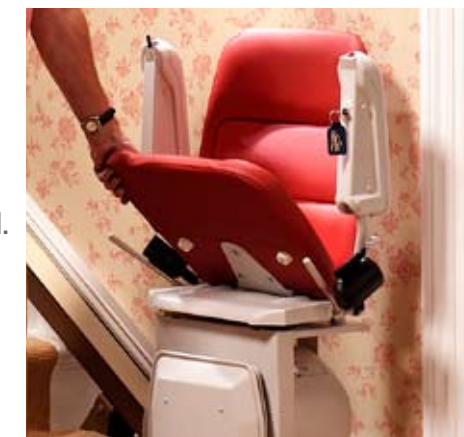

### Directional controls

can be positioned on either arm. Choose to either have a joystick or optional push-button controls.



### Swivel seat

is operated by easy-to-use levers on either side of the stairlift, allowing you to get off the chair with maximum ease. You can also choose a powered swivel seat that is operated via the joystick or push-button controls.





1. Unfolding the Saxon is exceptionally easy. Pull down the seat, followed by the chair arm and the arm-to-footrest link will automatically lower the footrest down too.



2. Once unfolded, make yourself comfortable in the seat. Use the safety belt for extra security if you choose.



3. Gentle pressure on the directional controls positioned on the arm is all it takes to make your stairlift move smoothly up or down the stairs.



4. When you reach the top of the stairs, you can swivel the chair round to face the landing so you can get off in complete safety.



5. You can leave the stairlift in its swivelled position at the top of the stairs as an effective safety barrier or fold it flat against the wall.



6. When not in use, the stairlift can be folded flat against the wall so it takes up less space in your home and leaves the stairs clear for other users.

**Stannah contact details:**  
**Stannah Stairlifts**  
**Watt Close**  
**East Portway**  
**Andover**  
**Hampshire**  
**SP10 3SD**

**Tel: 0800 715358**  
**Fax: 01264 338043**

[www.stannahstairlifts.co.uk](http://www.stannahstairlifts.co.uk)

Our company policy is one of continuing product improvement. We therefore reserve the right to alter the product specifications and to change and improve design at any time. Reconditioned products may have different specifications than those illustrated depending on availability. Full operating instructions are supplied for all products. Stannah is a registered trademark. The CE mark indicates that this product complies with the applicable European directives which relate to health, safety, environment and consumer protection. Product conforms to BS5776, in full (no exemptions). As with any other lift on whose reliability and safety you rely, we recommend that your Stannah stairlift is serviced annually by a trained engineer.

